



Maggie®

Two Step Cartridge-Based Solution



Volume 
 Viscosity 

5.0 mL

| | |
|-------------------------|--|
| Device Type | Disposable autoinjector |
| Primary Container | 5.0 mL cartridge with pre-installed needle (NIT®) |
| Dosage Type | Fixed dose Optional automated priming and titration |
| Route of Administration | Subcutaneous Intramuscular |
| Activation Method | Needle cover |
| Injection Feedback | Continuous audible clicks during injection, viewing window |

Copyright © 2024 SHL Medical AG. All rights reserved. Version 1.0 Information accurate at time of print and is subject to change without notice. Contact info@shl-medical.com for more information.

Handling Instructions

* For suspension drugs that require a priming step – hold device with cap upright during cap removal.



1
 Unscrew cap until needle cover automatically extends*. Then remove cap.



2
 Press and hold against injection site to activate.



+
 Discard in sharps container.

Key Features

- ✓ Simple 2-step operation
- ✓ Needle cover activation
- ✓ Continuous audible and visual feedback
- ✓ One-handed operation
- ✓ Permanently hidden needle
- ✓ Compact design
- ✓ Customizable needle length and gauge
- ✓ Passive locking needle cover
- ✓ Customized to client specifications